# Invacare® H440 Omega Eco

- EN** Commode chair  
User Manual
- DE** Toilettenstuhl  
Gebrauchsanweisung
- FR** Chaise toilettes  
Mode d'emploi
- IT** Sedia da comodo  
Istruzioni per l'uso
- ES** Silla WC  
Manual de instrucciones
- NL** Toiletstoel  
Gebruiksaanwijzing
- DA** Toiletstol  
Brugsanvisning
- FI** WC-tuoli  
Käyttöohje
- NO** Toalettstol  
Brukerveiledning
- SV** Toalettstol  
Bruksanvisning
- PT** Cadeira de WC  
Manual de instruções

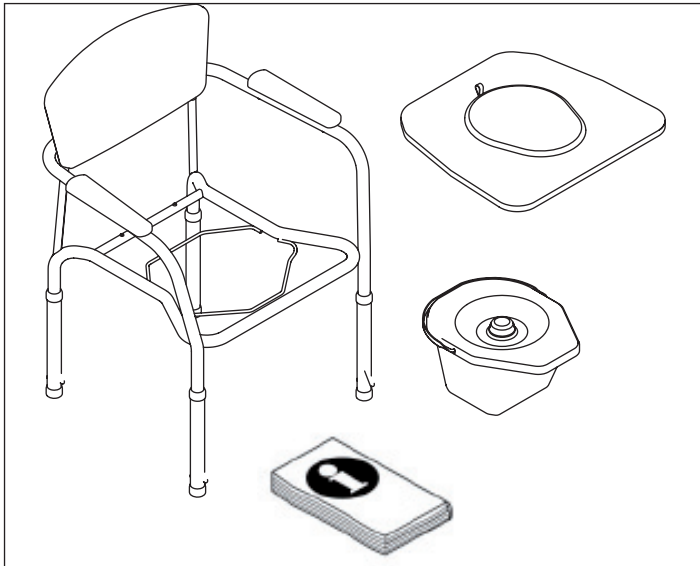
Fig. 1

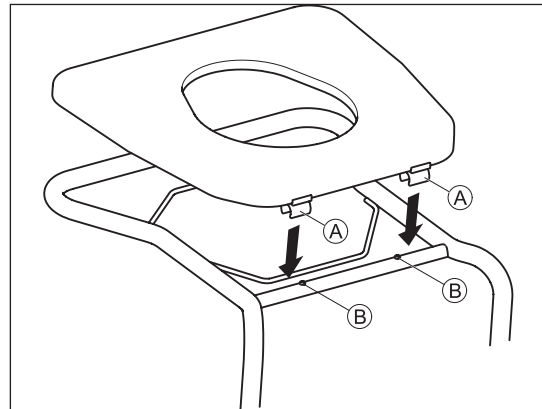
Fig. 2

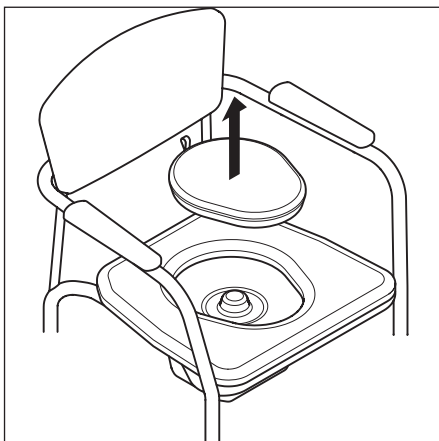
Fig. 3

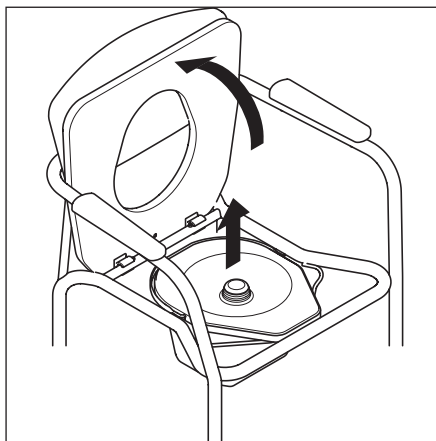
Fig. 4

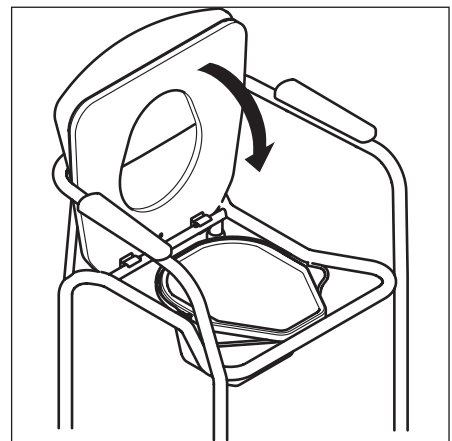
Fig. 5

**INVACARE** AQUATEC Operations GmbH  
Alemannenstr. 10  
D-88316 Isny

**H440 Omega Eco**

2012-02 140 kg

4 028698 114478

CE

LN (01)04028698114478(10)12BHS0001

INVACARE logo and icons for user manual and safety.

This manual must be given to the user of the product. Before using the product, read this manual and save it for future reference!

Yes, you can.

## Intended Use

The commode chair is to be used solely as an aid for going to the toilet. Any other use is prohibited.

### **WARNING** Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.

Temperature for use: 10 - 40 °C

Max. user weight: 140 kg

## Service Life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## Safety

### **WARNING** Risk of injury!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Protect the product against exposure to heat.

## Scope of delivery

- Fig. 1.

## Assembly

### **CAUTION** Risk of pinching!

- Take care during assembly.

### **WARNING** Risk of falling!

- Ensure that all legs are adjusted to the same height.
- Ensure that the pins are correctly engaged after adjustment.

- Fig. 2.

### **WARNING** Risk of falling!

- Ensure that, externally in each case, the clips Ⓐ engage next to the screws Ⓑ on the seat plate.

## Usage

### **IMPORTANT**

- Do not use the product in the shower.
- Use the product only at room temperature.

- Fig. 3 - 5.

## Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

- Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

### **IMPORTANT**

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control [www.nice.org.uk/CG002](http://www.nice.org.uk/CG002) and your local infection control policy.

- Clean by hand.
- Clean without scouring agent.

## Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined above are adhered to.

### **WARNING**

- Check product regularly for damage and to ensure that it is securely assembled.

## Re-Use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly and check the safety functions, and reset if necessary.

## Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulations.

## Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business. Guarantee claims can only be made through the relevant specialist dealer.

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Invacare A/S, Sdr. Ringvej 37,
Invacare AB, Fagerstagatan 9,