
Yes, you can.®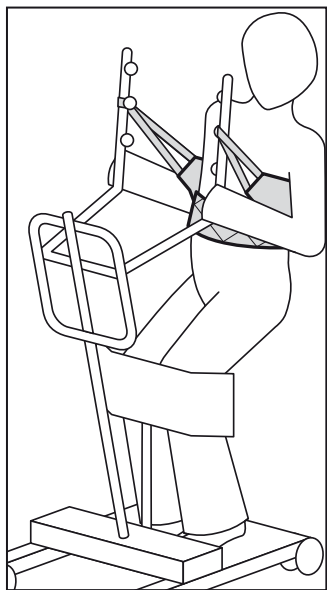

Invacare® Stand Assist

User's Manual (GB)

Brukerveiledning (NO)

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


Stand Assist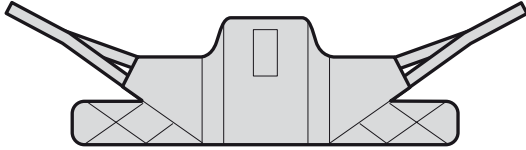


1.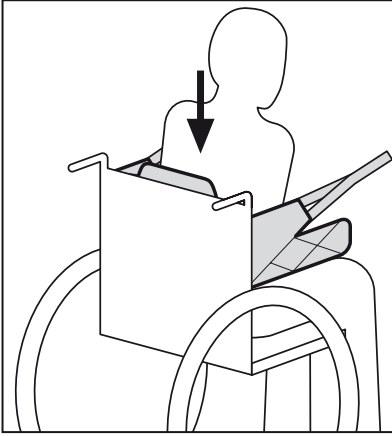


2.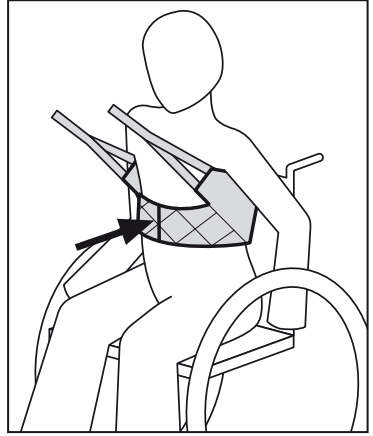


3.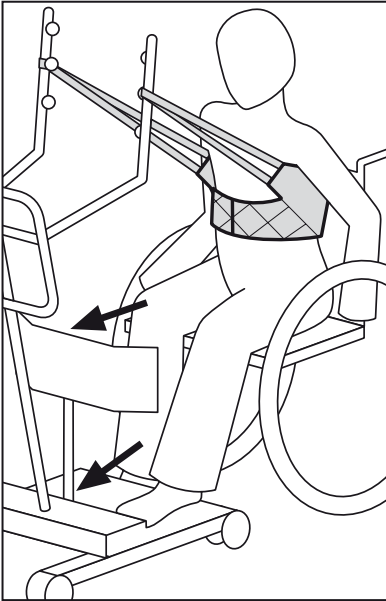

4.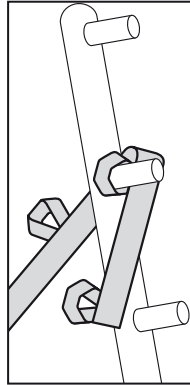

5.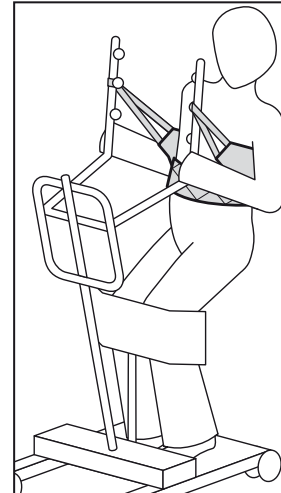

Quality Declaration

Congratulations with your new Invacare® sling from Invacare® EC-Høng A/S.

Your new sling is **CE**-marked in accordance with directive 93/42/EEC concerning medical devices.


Invacare® slings are developed and constructed with consideration for the user and others handling or assisting with lifting. Each sling type has been developed in accordance with the European Standard EN 10535.

Invacare® slings have been supervised and quality controlled throughout the entire production process, and the finished sling has been inspected by our finished goods control.

Please read the entire user's manual before using the sling.

Invacare® EC-Høng A/S is certified according to ISO 9001 and ISO 13485.

For and on behalf of Invacare EC-Høng A/S

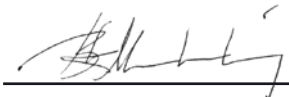Brian Mundeling
R & D Manager

Invacare® Stand Assist

USER'S MANUAL (GB)	2 - 3
BRUKERVEILEDNING (NO)	4 - 5
BEDIENUNGSANLEITUNG (DE)	6 - 7
GEBRUIKSAANWIJZING (NL)	8 - 9
MANUEL D'UTILISATION (FR)	10 - 11
MANUAL DEL USUARIO (ES)	12 - 13
MANUAL DE UTILIZAÇÃO (PT)	14 - 15

Stand Assist

Thank you for purchasing an Invacare® sling.

The following user's guide must be read and adhered to, or the warranty may be void. Please check your sling on delivery – if the sling is not in perfect condition, please contact your supplier.

The Invacare® Stand Assist sling used in conjunction with a standing hoist can provide an excellent transfer solution for those that have the ability to weight bear but are unable to independently stand. It also promotes independence and dignity of the client and allows a simple procedure for the caregiver.

The Stand Assist sling is suitable for clients with a significant trunk and head control and with an adequate arm and hand dexterity to hold the necessary bar of the hoist to stabilise themselves during the lift. The weight bearing ability should be bilateral.

Sling Application – Sitting in chair, wheelchair or on the edge of the bed

Lean the client forward ensuring that they are well supported to reduce the risk of falling and to promote feelings of security.

Slide the sling down behind the back of the client in a central position aligned with the waist.

Sit the client back against the sling. **Picture 1.**

Fasten the vest round the clients waist by using the adjustable buckle and ensure that the clients arms are outside the Stand Assist sling. **Picture 2.**

Place the clients knees secure against the kneepad and make sure that the feet are positioned well on the footplate. **Picture 3.**

Attach loops to the bar

Determine the best attachment point for the Stand Assist sling according to the height of the client. For a tall client the attachment point should be closest to the client and visa verse.

Note: The colour coded straps can be used to place the client in various positions. The colour makes it easy to connect both sides equally.

Attach the sling to the attachment point from the outside of the stand assist. **Picture 4.**

Make sure the following:

- The sling is wrapped around the outside of each lift arm.
- Use the closest colour coded strap to the client that maintains stability and comfort.
- The colour coded strap sits flush against the attachment knob.
- The same colour coded point is used on each lift arm.
- Always use the same colour code on each side of the Stand Assist sling.

Before lifting the client always instruct the client to do the following:

- Hold on the handgrips on both sides of the stand assist.
- Lean back into the Stand Assist sling. **Picture 5.**

Use only CE marked spreader bars and hoists, preferably with a rated Safe Working Load equal to or higher than that of the sling. If the SWL is lower than of the sling, the SWL of the spreader bar or hoist is applicable.

Always check that all the straps are properly attached to the spreader bar or lifting straps, before attempting to lift the client.

Invacare® recommend that a risk assessment is always carried out by a professional prior to issuing lifting equipment. Always choose the sling design and size according to the clients weight, size and physical ability.

Slings should be checked regularly for wear and damage* – NEVER use a damaged sling. (*In accordance with each country's regulations).

If you are in any doubt, please do not attempt to use the sling and seek the advice of a professional or your supplier.

Permissible max. load 159 kg.

Symbols:  = 159 kg     

Safe Working Load, consult the Users Manual, do not exceed 95°C washing temperature, do not bleach, do not tumble dry, do not iron.

All Invacare® slings are guaranteed for one year.

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MOBITEC MOBILITÄTSHILFEN GmbH
INVACARE EC-Høng A/S